


CHEETAH INSTRUCTIONS

GETTING TO KNOW YOUR SPEECH BUDDY™

Give the patient a chance to familiarize themselves with their **Speech Buddy™**. Allow the patient to handle their **Speech Buddy™** themselves, point out the target to them and let them touch it with their finger.

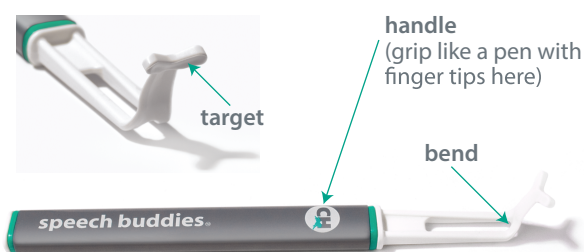


HOW TO POSITION YOUR /ch/ SPEECH BUDDY™

Grip the handle of your **Speech Buddy™** like a pen. Insert the tip of your **Speech Buddy™** into the patient's mouth. Position the bend of the **Speech Buddy™** on the bottom of the two front teeth.

Ensure that the handle is angled appropriately to bring the target against the roof of the mouth. Adjust handle angle as needed to place and maintain the target in position.

Pull slightly back against the back of the teeth and use just enough pressure to hold the **Speech Buddy™** in position.

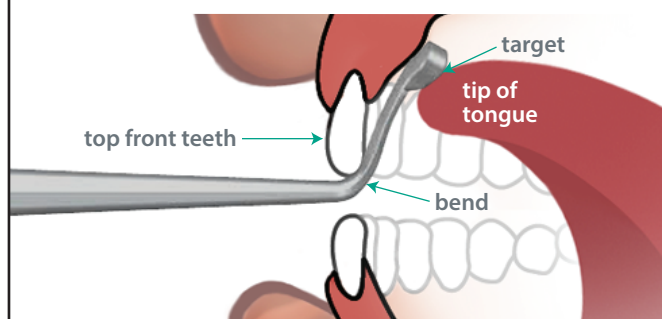

CH USING YOUR /ch/ SPEECH BUDDY™

The first step towards proper articulation of the **/ch/** sound is to achieve a correct production of the sound with the **Speech Buddy™** in place. Once the **Speech Buddy™** is in place, confirm that the patient can feel the target with the tip of their tongue. Have them respond with a simple yes or no.

Instruct the patient to first make contact with the target using the top portion of the tongue near the tip of the tongue. Then instruct the patient to say the **/ch/** sound while trying to contact the target.

Once a correct production has been achieved, alternate prompting consecutive productions with and without the **Speech Buddy™** as needed.

Start first with sounds in isolation, then sounds in simple syllables (e.g. **cha**). Next move to words with the **/ch/** in initial position (**che**etah), medial (mat**ch**ing), final position (wat**ch**), and finally to phrases.

clear speech is close at hand



SPEECH BUDDY™ IN THERAPY AND AT HOME

Your **Speech Buddy™** has been designed by speech therapists for speech therapists. **Speech Buddies™** can be used in conjunction with your therapist's favored approach for a given patient.

It can also be used by parents. Using the **Speech Buddy™** at home can enable your patient to have more opportunity to practice between therapy sessions and can expedite their learning.

Go to www.speechbuddy.com for more information including a video tutorial.

WARNINGS

- DO NOT use the **Speech Buddy™** on multiple patients
- DO NOT use if the **Speech Buddy™** or package is damaged

PRECAUTIONS

- DO NOT allow patient to chew on the **Speech Buddy™**
- DO NOT allow patient to put the handle of the **Speech Buddy™** in his or her mouth
- To wash your **Speech Buddy™**, use mild soap and water or dip quickly in 70% isopropyl alcohol. Rinse thoroughly with water.
- Store in a cool and dry place. DO NOT expose to temperatures in excess of 110°F (43°C) or below 32°F (0°C)

INTENDED USE

- Exerciser, Non-Measuring

MANUFACTURED BY
Articulate Technologies Inc.
3150 18th Street, Suite 340
San Francisco, CA 94110
United States
T: 415.977.9038 F: 415.358.4296

Speech Buddies™ is a trademark of Articulate Technologies, Inc.
Patent pending. © 2013 Articulate Technologies, Inc. LB002.C

